MAWe
by GADNIC

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BEBE0013

Índice

Español	5
Português	15
English	25

Núcleo inteligente

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Intelligent core Integrated electric breast pump


Instructions for use

1. INTRODUCTION


This automatic breast pump has a unique design that allows you to maintain a more comfortable milking position. The soft massage cushion design provides a soft and warm feeling and can mimic baby's sucking action so that breast milk can come out quietly, gently and quickly. The pump is compact and easy to assemble. All parts can be safely cleaned in the dishwasher. The breast pump is Bisphenol A free. Breastfeeding experts say that breast milk is the best nutritious food for babies under one year of age. After six months of life, babies can continue to breastfeed and gradually be fed with complementary foods. Breastmilk is especially suited to a baby's needs, as it contains antibodies that protect against infections and allergies. Breast pumping can help to prolong breastfeeding time. You can express and store breast milk, so even if you cannot breastfeed yourself, your baby can still enjoy breast milk. As the breast pump is compact and stays neatly tucked away when expressing milk, you can always carry it with you so you can express breast milk for your baby at any convenient time.

Main engine schematic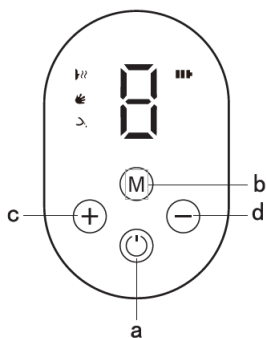


- a. ON/OFF
- b. Change suction mode
- c. Increase suction level
- d. Decrease suction level

Liquid crystal diagram as follows: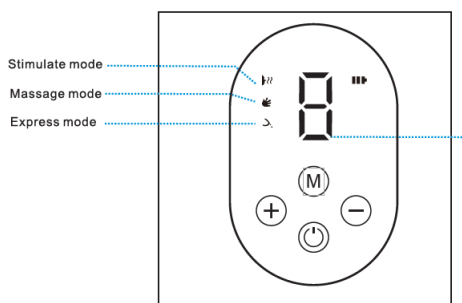

2. PARTS GUIDE

1. Dust cap
2. Massage cushion
3. Connector
4. Silicone diaphragm
5. Valve
6. Baby bottle
7. Teat
8. Pump motor
9. Cable and adapter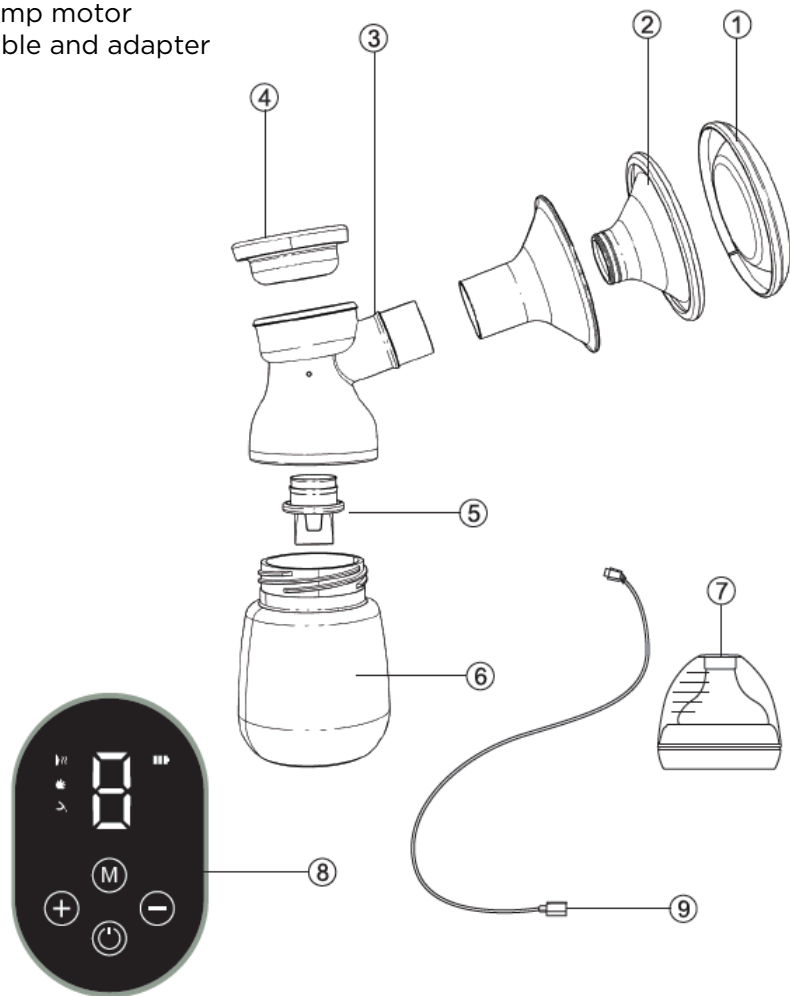

3. PREPARATION AND ASSEMBLY

Before first use, clean and disinfect all components according to the chapter 'Cleaning and disinfection'. Disinfect all components before each use.

BREAST PUMP ASSEMBLY


Note: Be sure to sterilise all components of the breast pump and wash your hands thoroughly before touching clean components.

Please note that clean components may still be hot.


Before assembly, disinfect all breast pump components and wash your hands thoroughly.

Tip: You may find it easier to fit the breast pump when it is wet.


PLEASE READ THE FOLLOWING INSTALLATION DIAGRAM CAREFULLY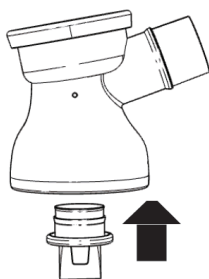

1. Insert the valve into the bottom part of the connector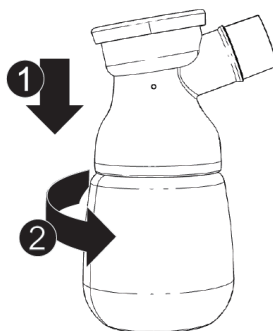


2. Screw the connector onto the bottle until it is fully tightened.

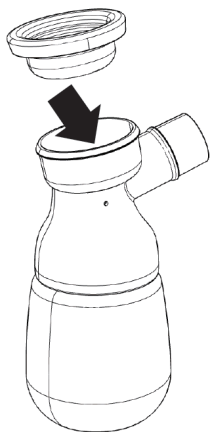3. Place the diaphragm on top of the connector. Press the diaphragm down to ensure stability.

4. Place the pump motor on top of the connector. Press it down to secure it.

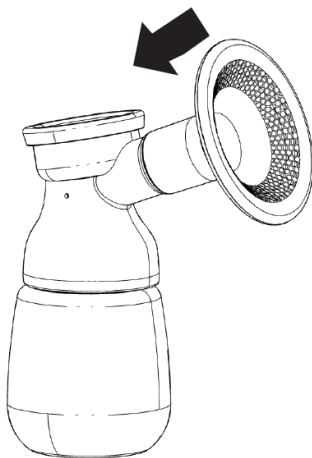5. Place the massage cushion on the funnel part of the breast shield, press and make sure the cushion fits perfectly, press the petals to remove residual air, finally connect the power adapter to the motor.

4. PROBLEM SOLVING

Lack of suction	Check the correct installation of the spout valve of the pump assembly for damage (note when cleaning not to rub) to ensure that the silicone diaphragm and petal pads are firmly attached to the main body of the milk suction device to achieve the ideal sealing state.
Unable to suck milk	Unable to express milk to ensure proper suction and complete breast pump attachment. Relax and try again. After repeated attempts you may be able to express milk. See tips for successful suction on page 6.
Chest pain when using	Avoid applying excessive force when using the device, as it is not necessary to generate all suction immediately. It is recommended to start with massage stimulation mode to prepare the breast and then switch to milk expression mode. If in doubt or in need of guidance, consult a healthcare professional or breastfeeding specialist.

5. USING THE BREAST PUMP

When to express breast milk

It is recommended (unless experts say so) to wait for breastfeeding and feeding time to normalise (at least 2-4 weeks after the birth of the baby).

Exceptional cases

You have to breastfeed your baby in hospital.

When you have breast pain (soreness or swelling), you can express a small amount of milk before breastfeeding or between feedings to relieve the pain and help your baby suckle more easily.

If your nipple is sore or cracked, you can use the breast pump once it has healed.

If you are temporarily separated from your baby, but want to continue breastfeeding during the reunion, you should express milk regularly to stimulate milk secretion.

6. PUMPING TIPS

You should choose the best time of day to express breastmilk, for example: in the morning, when your milk is plentiful, before or after the first feeding or when the baby does not drink breastmilk after feeding. You can express milk during a break from work. You may have to try several times before you succeed in pumping. Fortunately, this automatic breast pump is easy to assemble and use, and you will soon be able to express milk.

Before using it for the first time, familiarise yourself with how it works and choose a time that is not rushed or disruptive. Pictures of your baby help to encourage the 'breastfeeding' reflex. A warm environment also helps to express milk - try expressing after a bath, or put a warm towel on your breast for a few minutes before expressing. Your baby may find it easier to suckle when suckling at the other breast or after breastfeeding.

If you feel pain during the process, stop and consult breastfeeding experts.

7. SIMPLE INSTRUCTIONS

- Press the power button to enter stimulation mode, press the mode button to enter express mode, then press again to enter massage mode.
- Massage mode (9 levels in total)
- Stimulation mode (9 levels in total)
- Express mode (9 levels in total)
- For each press, the display shows the current level.
- The machine switches off automatically in 30 minutes by default.
- The battery display will flash when the battery is charged. After full charge, it will display the full grid. The milk in the bottle should not be too full to overflow the relief valve.

8. INSTRUCTIONS FOR USE

1. Wash your hands thoroughly and make sure your breasts are clean. Relax and sit in a comfortable chair (you can use a cushion to support your back) Make sure you have a glass of water next to you.
2. Gently support your breast, place the cover and make sure your breast is centred, and the breast massage cushion forms a seal.
3. Press the on/off button, the display lights up after start up, the breast pump starts with breastfeeding mode automatically, press + - to choose the massage strength, when milk starts to come out, press the mode conversion key to switch to pumping mode, choose the most comfortable suction strength and frequency for pumping.
Note: it is not necessary to use the biggest suction, just use the suction you feel comfortable.
4. Once pumping is complete, turn off the breast pump and carefully remove the main body of the breast pump.
5. Unscrew the bottle from the main body of the breast pump to prepare it for feeding/storage. Follow the instructions in the chapter 'Cleaning and disinfection' to clean other used parts of the breast pump.
6. Unplug the main motor component.

9. CLEANING AND DISINFECTION

Before first use, clean and disinfect the breast pump.

Clean all parts each time you use it. Completely remove the breast pump as well as the relief spout.

Be careful when removing and cleaning the valve. If the valve is damaged, the breast pump will not work properly. To remove the valve, gently pull the small side of the valve. Clean all parts with warm water and a little detergent, then rinse well.

To clean the valve, gently rub it in clean water with a few drops of cleaning fluid, do not insert any objects into the valve. Use a special sterilizing container to disinfect all parts or boil them in boiling water for 5 minutes.

10. STORAGE AND HEATING

Conservation of breast milk.

When you finish expressing milk, cover or seal the storage bag, refrigerate or freeze it immediately. the refrigerator should be refrigerated. Breast milk can be kept cold for 48 hours and in the freezer for 3 months.

Heat the milk.

After thawing, warm breastmilk with a milk warmer or warm water - do not use a microwave or boiling water to warm breastmilk, as this can cause loss of nutrients. Warmed breastmilk can feed your baby well.

ATTENTION

1. If you want to freeze breastmilk, do not fill the container or bag completely. Fill it 3/4 full, leaving enough room for expansion when freezing. Avoid placing breastmilk in the refrigerator door, as temperature fluctuations may affect its quality.
2. To preserve the nutrients in breastmilk, thaw it in the refrigerator overnight. When warming is necessary, use warm running water (below 37°C) to warm the bottle or bag.
3. Gently shake the bottle or pouch after warming to mix the separated fat and ensure an even consistency of the milk.
4. Do not heat frozen breastmilk in the microwave or in boiling water, as this may destroy essential nutrients and increase the risk of burns to the baby.
5. Thawed breastmilk should not be refrozen. Also, do not mix fresh breastmilk with milk that has already been thawed.

DANGER

This product is a personal item, do not share it, so as not to affect health. Be careful when removing or connecting parts, do not use excessive force. This product should not be exposed to hot sun and should be placed where children cannot reach it.

Please note that it is strictly forbidden to wash the main engine.

Warranty

Loss of warranty:

1. Careless or improper use and damage caused by misuse, disqualifies the warranty.
2. Lost or damaged purchase invoice or after-sales service card.
3. Damage caused by natural disasters.

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