# MAWe

by GADNIC

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**BEBE0015**

## INDICE:

|           |    |
|-----------|----|
| ESPAÑOL   | 02 |
| ENGLISH   | 12 |
| PORTUGUÊS | 22 |

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
# **ELECTRIC BREAST PUMP**

Manual for use of electric breast pump  
Single bilateral universal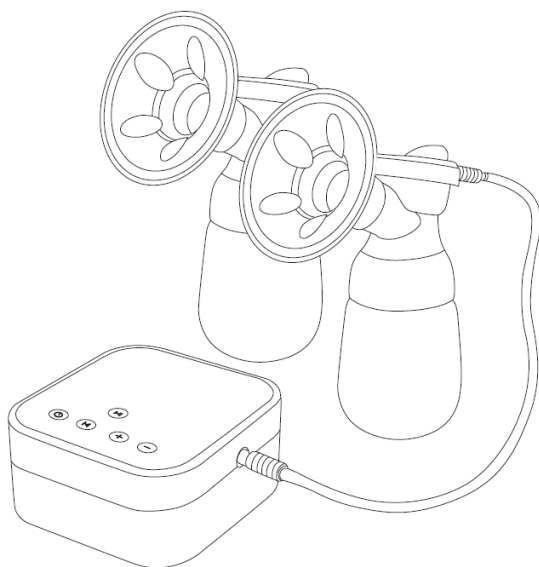

PLEASE READ THIS INSTRUCTION CAREFULLY BEFORE USING THE PRODUCT AND  
KEEP IT FOR FUTURE USE.

## CONTENTS

|                                 |    |
|---------------------------------|----|
| Product introduction            | 01 |
| Description of components       | 02 |
| Installation instructions       | 03 |
| Description of components       | 04 |
| Troubleshooting                 | 06 |
| Maintenance                     | 06 |
| Matters needing attention       | 07 |
| Statement of conformity         | 07 |
| Hazardous substance declaration | 08 |
| After-sales service             | 09 |

## PRODUCT INFORMATION

Model:

Product name: electric breast pump

Main materials: ABS, the silicone, PP

Technical parameters:

Charging input voltage: 5V

The lithium battery: 3.7V; 2000mAh  
1600mAh(RH-298)

Power rating: 5W

Noise limit : 50dB(A)

Motor protection : 11

Conditions of use : 5°C~40°C, 10%~90%RH

Functional features

Product features:

1- Automatic simulation: Controlled by a microchip program, it imitates the intermittent breastfeeding of infants.

2- In galactagogue mode, the intensity can be adjusted to 9 levels, In the massage mode, the intensity can be adjusted in 9 levels to effectively massage the breast; In the breastsucking mode the intensity can be adjusted in 9 levels, which imitates the speed of the infant's sucking rhythm and the size of the infant's sucking volume.

Product features:

1- The cylinder design with gas-liquid separation structure, no milk return when sucking.

2- The removable liquid silicone horn imitates the principle of ergonomics.

It can suck and massage, squeeze the breast and stimulate lactation when sucking


3- Touch design of control panel, interface display, clear operation process.

4- The mainframe has built-in battery, which can be easily used when power is off or when going out or at home.


Parts introduction

| Part Name            | Quantity | Material | Part Name         | Quantity | Material      |
|----------------------|----------|----------|-------------------|----------|---------------|
| Dirt shroud          | 2        | PP       | Silicone speaker  | 2        | Silica gel    |
| Horn Mouth           | 2        | PP       | Silicone cylinder | 2        | Silica gel    |
| Three Links          | 2        | PP       | Cylinder head     | 2        | PP            |
| Through Joint        | 4        | PP/TPR   | Flexible pipe     | 2        | Silica gel    |
| Cable                | 1        | /        | Duckbill valve    | 2        | Silica gel    |
| Main Engine          | 1        | /        | Pacifier kit      | 2        | Silica gel+PP |
| Wide bottle of 180ml | 2        | PP       |                   |          |               |


Warm Tip : before each use. should be pre-sterilized with breast milk has direct contact with the components: bottle, silicone horn, three-way silicone cylinder, duck mouth valve.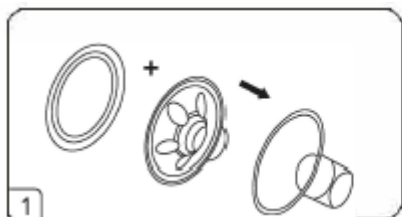

Put the silicone horn and bell mouth in sequence, then put in the dust shield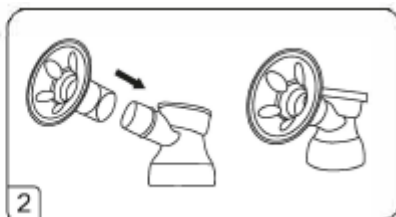

Insert the silicone horn and bell mouth assembly into the three links to ensure that the sleeve is tight

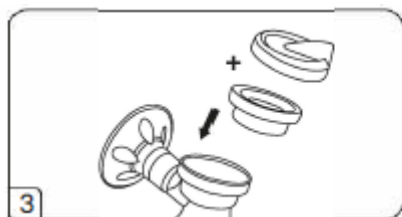
Put the cylinder and cylinder head into three links and tighten the cylinder head.

Put the duckbill valve into the bottom outlet of the three-way to ensure that there is no leakage.

Screw the three-way assembly into the bottle.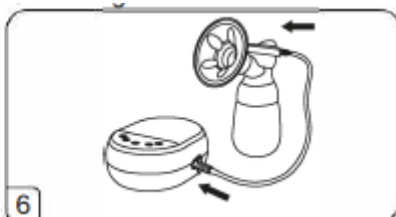


Insert the straight-through joint of the hose into the cylinder head and the relative hole position of the main engine to ensure that there is no air leakage and can be used.

Note: the through joint should be opened before insertion into the main airway hole; do not start the main engine work without opening the dust shield, otherwise it will affect the service life of the main engine.

Ready for use

- Before use, check and ensure that the sterilized parts (parts in direct contact with breast milk) are clean and water free, and the absorber is firmly connected to the mainframe.
- Clean, compress and massage breasts and areola with wet and hot towels before sucking. Relax your body to relieve the pain of bulging milk.

Panel operating instructions

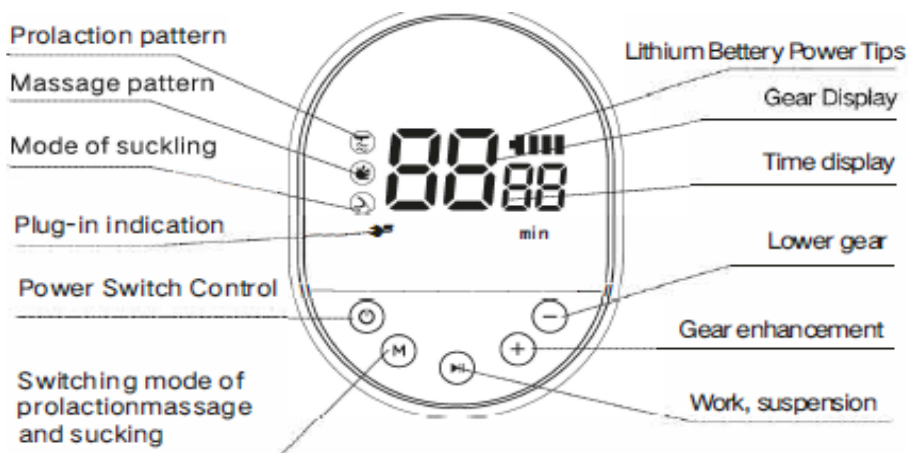Boot mode switch

- Touch “power Switch Control” for 2 seconds and enter prolaction mode by default.
- After boot can carry on prolaction, massage, sucking mode switch.

## Screen Display

When the working mode is prolactin. light is on; when the working mode is "massage", the light is on; when the is on, it is working mode; when the x is on. it means shutdown

- When "prolactin" mode, the intensity can be divided into 9 files, the screen will also display the corresponding number.
- "Massage" mode, there are 9 stalls, the screen will display the corresponding number
- In suckling mode, there are nine stalls and the screen displays the corresponding numbers
- When supplied by the built-in battery, the on the LEO screen is a static display of electricity; when supplied by the charging line, the battery charging state will be dynamically displayed in the shutdown state
- When the battery power drops to the lowest, the right side of the battery symbol flashes, the other three are not bright, and the LEO screen goes out after 10 seconds.

## Frequency regulation of suckling strength

- "Breast" mode, touch + and - can adjust the strength of milk, a total of 9
- "Massage" mode, press + and - can adjust the intensity of prolactin, a total of 9 stalls.
- "suckling" mode, by + and - can adjust the strength of suckling, a total of 9 stalls.
- At work, pressing the M key can freely switch the working mode. When switching to "prolactin" mode, the default is the 1st gear used; when switching to "suckling" mode, the default is the 1st gear used.

## Suspension function

- Press pause mode, machine tentative, press open again, shut down can not open.

## Memory function

- Start prolactin, massage suckling mode, adjust the required gear run for more than 3 seconds it will automatically remember. The next start-up will start the above run gear.

## Battery power charging

- In the working state, the power adapter is connected and supplied by the power adapter. When the power adapter is not connected, the host will automatically use the built-in battery to supplied power.
- When shut down, the power adapter can charge the battery; after full charge, it can work continuously for more than 1 hour.
- When the power is too low, the host automatically stops working and prompts to protect the battery; when the power is full! automatically stops charging to avoid damage to the battery.

## Malfunction Analysis

The following problems may occur during milk absorption, please refer to the method provided.

| Phenomenon    | Analysys of causes                                                                                                                                                                               | Processing method                                                |
|---------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------|
| Small suction | The connection is in the airleakage (duckbill v alve, siliconehorn, dirc-theac three-way and main engine), the main engine output air path is blocked, s uch as hose folding, hose airway debris | Check the connection parts and reinstall, clean and clean again. |
| No suction    | Duckbill valve not installed or deformed Misuse of plastic cornponents to disinfect cupboards, miaowave ovens and pressure cookers, resulting in deformation and damage                          | Install platypus valves or renew fittings                        |
| No suction    | The adapter is not con nected to the charging socket or main frame socket. Lithium batteries are running out                                                                                     | Re-plug the adapter of Charge it before using it                 |
| No power on   | Find milk or water entering the cylinder or hose to check if the cylinder is damaged or not in stal led                                                                                          | Stop immediately, clean or replace the cylinder and reinstall    |

## Maintenance

- Battery is built-in non-detachable do not disassemble
- Host or battery charging or power supply requires a specified DC5V power adapteror mobile phone power adapter.
- Do not use other non-standard power adapters tocharge or power
- Do not approach the high temperature or fire source during the use and placement of the not place mainframe in heater or fire source to avoid combustion and explosion
- When the host is not used for a long time, it is recommended to charge once a month.
- If there is a failure in the mainframe, please do not disassemble and repair it. It mustbe sent to the designated maintenance point of our company for repair orreplacement by professional personnel.
- Before use, the accessories directly in contact with breast milk(bottle, silicone horn,three way, cylinder, duck mouth valve, milk bottle cap) should be cleaned anddisinfected in advance, can be sterilized by steam or hot water, and the disinfectiontime should not exceed 1 minute
- The host can't put it in water

## Attentions

1. Before use, remove all packaging of this product, check the complete parts against the product specification
2. This product is a personal special product, do not use many people, lest affect health
3. When cleaning this product, please pull the adapter out of the power outlet, do not use bleach, chemical disinfection solution, tablets, corduroy, abrasive cleaner or corrosive liquid (such as gasoline, acetone or alcohol) to clean this product.
4. When removing or installing all kinds of components, such as three links, silicone horn, bell mouth, silicone cylinder, duckbill valve and so on, we should be careful to operate, do not force too hard
5. Cylinder head can not be washed with water, cleaning such as water, dry water drop, dry rear can be used
6. In use, the hose must be clean without water droplets, otherwise it will affect the performance of the main engine.
7. It is recommended to use special steam or boiled water to disinfect the accessories that need to be disinfected. It is not allowed to use disinfected cupboards, microwave ovens or pressure cookers to heat and disinfect the accessories. The host machine is not allowed to enter the water for cleaning
8. Packaging and all components and mainframe are not toys. Keep out of reach of children.
9. Store in a cool and dry place, away from direct sunlight, away from fire and heat source
10. When it is difficult to suck milk, you should relax your body and then suck milk. When the swelling milk pain, should be hot compress massage; Consult your doctor if you feel uncomfortable or cannot pump milk for many times
11. If parts are damaged, to avoid danger, please stop using and send them to our service center for maintenance. Do not use accessories or parts produced by other manufacturer or without our permission to avoid danger.

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