MAWe
by GADNIC

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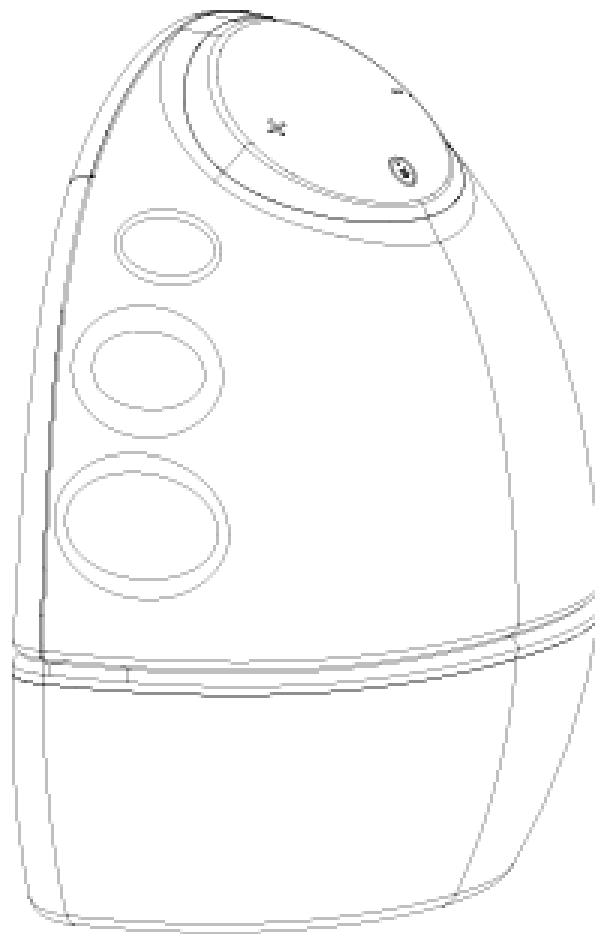
INSTRUCTIONS FOR USE

BREAST PUMP BEBE0155

PLEASE READ THE INSTRUCTIONS CAREFULLY BEFORE USE. KEEP THE MANUAL FOR FUTURE REFERENCE

Wearable Breast Pump

Provides a hands-free, more enjoyable, and reassuring way to pump breast milk. The breast pump is designed to be used wherever you are. It is small and lightweight enough to fit inside your bra.

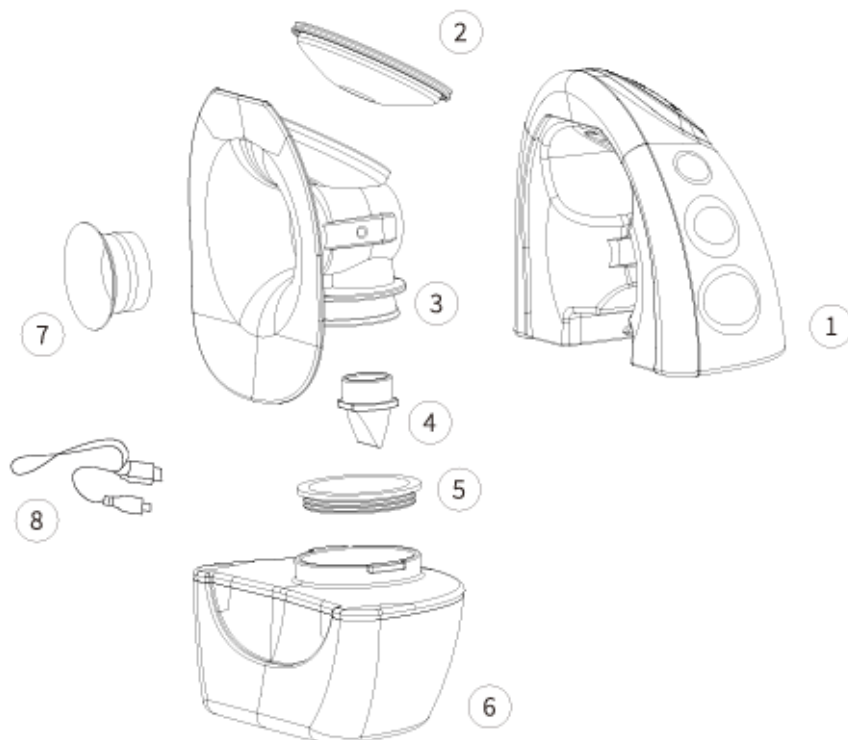
- Bluetooth intelligent connection control makes it more high-end, stylish, and convenient.
- Hands-free design completely frees your hands.
- No tubes, no wires, no swinging baby bottles.
- No fixed pumping schedule — pump breast milk anytime, anywhere as you like.
- Integrated device for easy assembly and cleaning.
- Made of food-grade materials, free of BPA (bisphenol A).
- Simple and direct installation; all accessories are detachable for easy cleaning.
- Four modes with 9 levels each, providing you with more usage choices.

Precautions for Use

1. Before using this product, carefully read the user manual.
2. Fully charge the device before initial use after purchase. (The device cannot be turned on during charging).
3. If the device is damaged, repairs or replacements must be performed by professionals from the manufacturer or authorized service centers to avoid hazards.
4. Do not expose the breast pump to high temperatures or direct sunlight.
5. Strictly prohibit throwing the main unit into fire.
6. Never immerse the main unit or power adapter in water or other liquids for cleaning.
7. Do not use, charge, or discharge the device near open flames or in environments exceeding 45°C (113°F).
8. Keep packaging bags away from children to avoid suffocation hazards.
9. Breast pump parts that directly contact breast milk are personal care items and should not be shared between mothers.
10. Before use, clean and sterilize accessories. After use, clean immediately and keep components dry. If no milk is expressed, do not continue suction for more than 5 minutes.

11. Avoid excessive suction to prevent breast injury.
12. If the silicone cylinder is damaged, stop using the pump immediately and replace it with a spare.
13. After cleaning and sterilizing the silicone cylinder, ensure its surface is completely dry before reattaching it to the main unit.
14. When charging, use an adapter with input AC 100-240V and output DC 5V, 1A.
15. This product is intended only for the purposes described in this manual.

Accessory Description

Serial Number	Part Name	Temp. Resistance	Quantity	Material
1	Main Unit Assembly	/	1	/
2	Silicone Cylinder	-20 - 200°C	2	Silicone
3	Horn Cover (Breast Shield)	-20 - 109°C	1	/
4	Duckbill Valve	-20 - 200°C	2	Silicone
5	Sealing Ring	-20 - 200°C	1	Silicone
6	Bottle	-20 - 109°C	1	/
7	Conversion Head	-20 - 200°C	1	Silicone
8	Charging Cord	/	1	/

Display and Operating Instructions


Symbol	Name	Description of Use
	Power / Mode Key	Long press 2 seconds to turn on the machine; it enters massage mode by default. Short press to switch modes (Massage, Breast Pumping, Enhanced, Mixed). Long press 2 seconds again to turn off. (Auto-off after 20 minutes) .
	Key +	1-9 levels; each press increases suction power by one level.
	Key -	1-9 levels; suction power decreases by one level for each press.
	Level Display	Displays the current level, 1-9.
	Massage Mode	Low suction, fast pace, levels 1-9.
	Pumping Mode	Normal rhythm, levels 1-9.
	Enhanced Mode	High suction, slow tempo, levels 1-9.
	Mixed Mode	Combined massage and suction rhythm, levels 1-9.
	Battery Display	Shows actual power. One flashing bar indicates low battery. Full icon stops flashing when fully charged.

APP Connection Instructions

1. First, download the app based on your phone's operating system: (search for "BreezeFeed" in the Apple App Store or Google Play Store).
2. After downloading and installing the app, grant the "Nearby Devices" and "Location" permissions as required.
3. Turn on Bluetooth on your phone.
4. Turn on the product.
5. Open the app, which will connect automatically. Once connected, you can control the product via your phone.
6. For future use, simply open the app.


Notas:

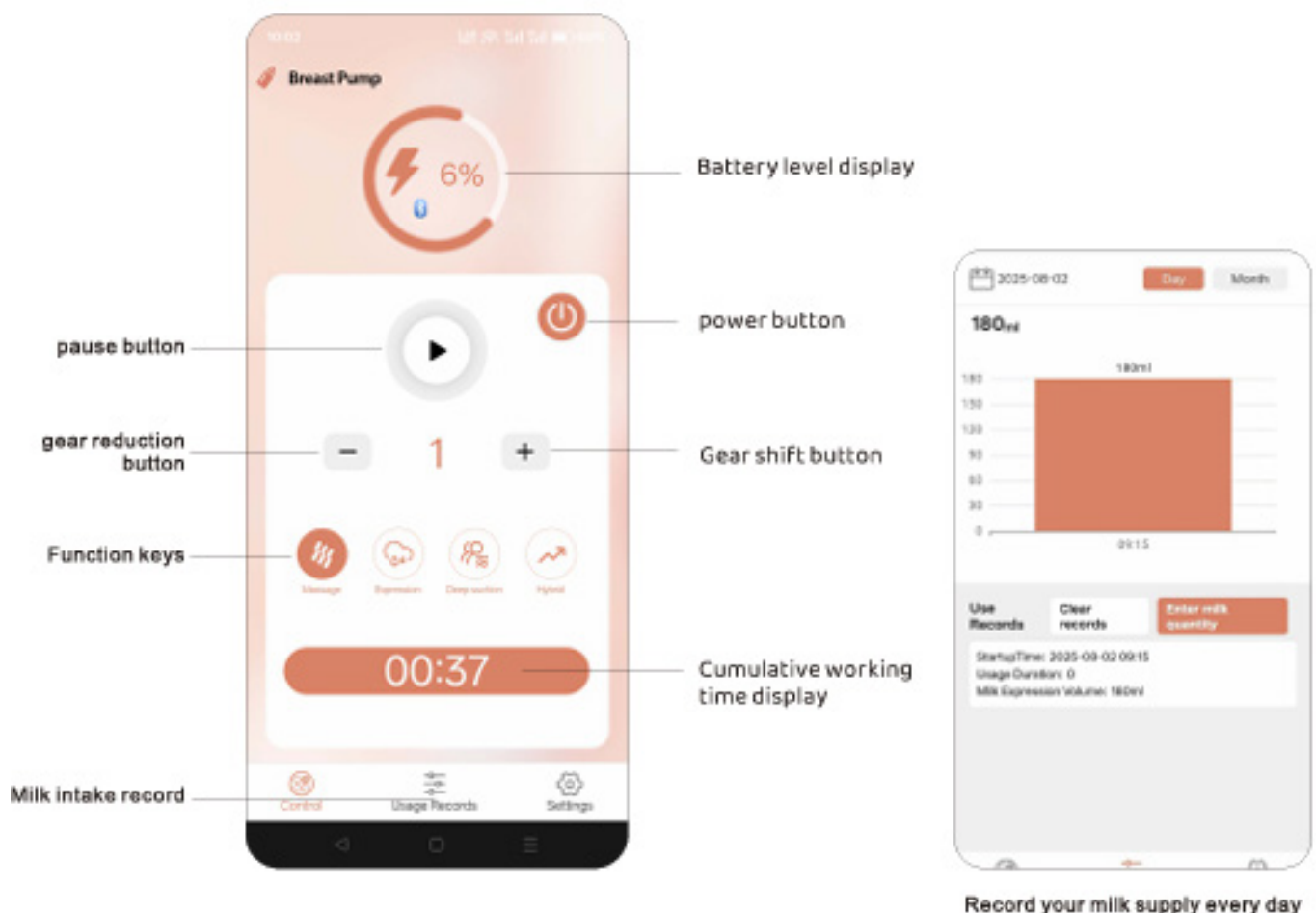- a. The distance between your phone and the product must be less than 1 meter to avoid Bluetooth signal interference.
- b. The product must be turned on to be controlled. The app cannot turn the product on; it can only turn it off or pause it.


Assembly Method

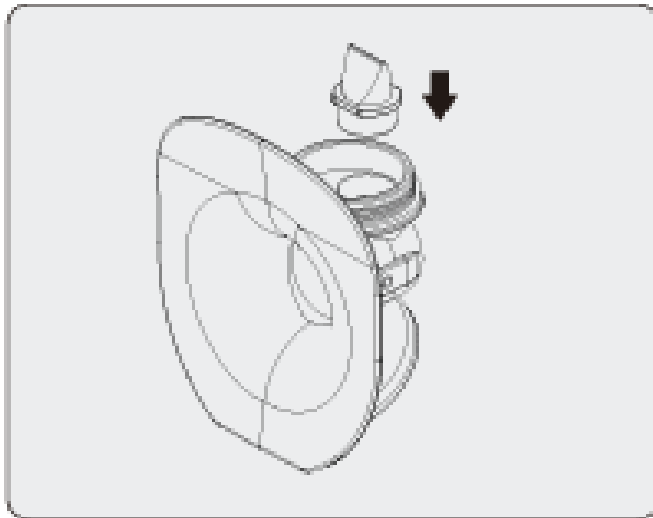
When using the product for the first time, please fully charge the product first.

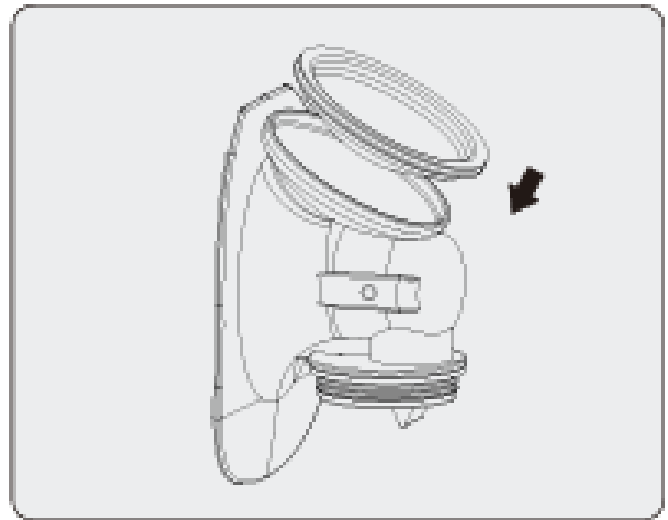1. Press the duckbill valve release direction into the round hole of the flapper cover on the picture, it needs to be pressed in place. (Pay attention to the direction of the duckbill valve)

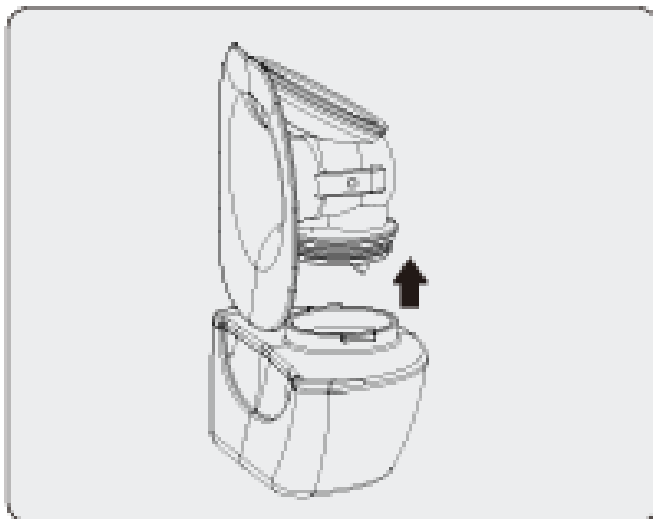2. Press the silicone cylinder into the cylinder groove of the horn cover, the edge of the cylinder and the edge of the horn cover should be pressed flat. (If not installed in place there may be air leakage)


3. Press the bottle onto the flapper in the direction.


4. Push the assembled speaker cover assembly into the host according to the diagram until you hear the "ticking" sound that is installed in place. (Assembly is complete, can be used)

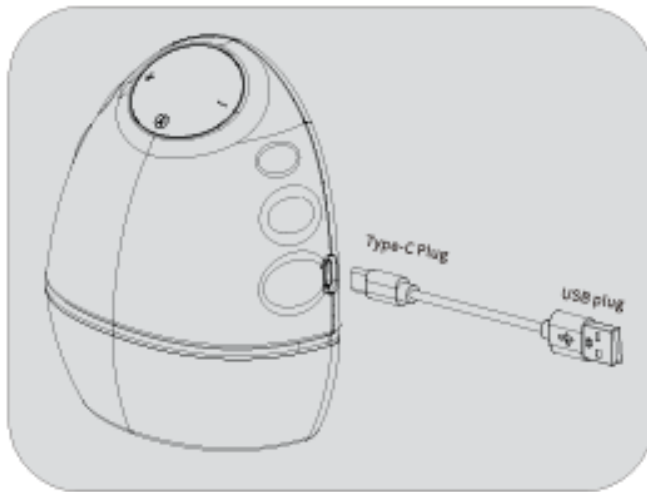Charging method


When charging, insert the Type-C plug end into the side of the host, insert the USB end into the adapter (DC5V 1A) or charging treasure, computer USB hole. When charging, the display power logo flashes, until the power full logo stops flashing.

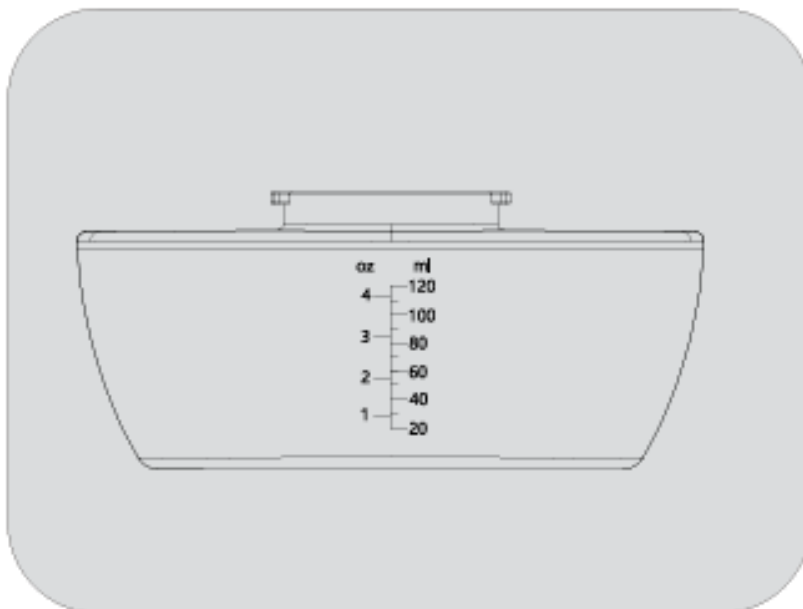Bottle Capacity Considerations


The milk in the bottle should not exceed the maximum capacity scale. If it reaches the maximum capacity scale, please switch off the breast pump immediately. (The main unit is set to 20 minutes continuous operation with automatic switch-off function)

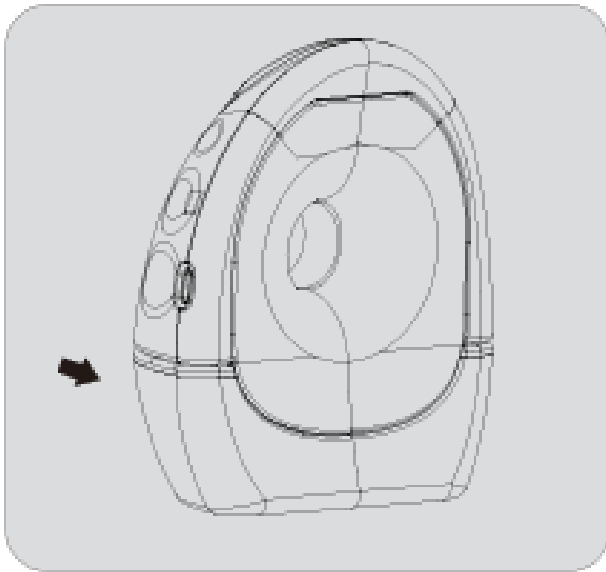How to disassemble the breast pump

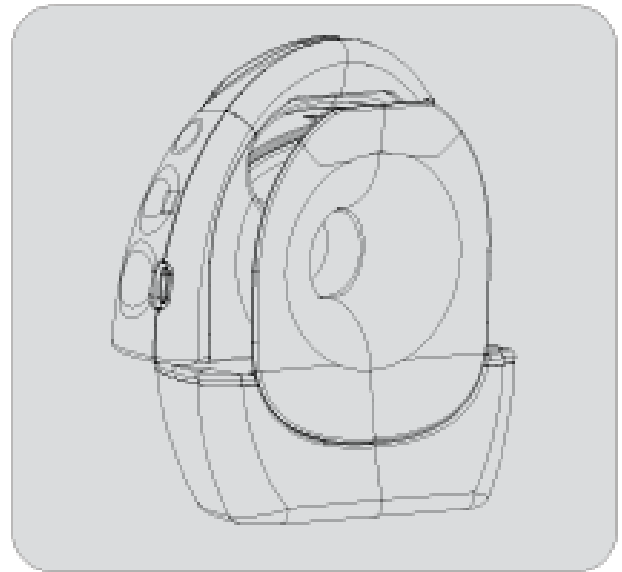1. Hold the main unit with your left hand and push the bottle forward with your right hand to remove the bottle assembly.

2. If there is milk in the bottle, pay attention to the whole machine shall not be skewed too much, so as to avoid milk leaking out from the vent.

3. Remove the bottle firmly downwards.

4. Remove the duckbill valve and silicone cylinder and clean, sterilize and dry for next use.

Operation method

1. Before use, please wash your hands, ensure that the breast pump components have been thoroughly cleaned, and double-check whether relevant accessories are installed in place.
2. Before breast pumping, apply a warm towel to the breasts for hot compress and massage, which will improve the pumping effect.
3. Align the middle hole of the assembled main body breast shield with the nipple, press it tightly against the breast, confirm no air enters, power on to ensure complete fit, and start using when obvious suction is felt.
4. After placement, adjust the underwear to be tight.
5. Adjust according to personal comfortable suction: Gradually increase the suction until slight discomfort (not pain) is felt, then slightly decrease the suction. You can also open the app to control and adjust modes and gears via mobile phone.
6. Maintain an upright posture or slightly lean forward during pumping for optimal milk flow into the bottle.
7. After finishing pumping, long-press the power button for 2 seconds to turn off the device and remove the breast pump.
8. Immediately clean the components in contact with milk stains after pumping.

Troubleshooting

Problem	Solution method
Feeling pain when using the breast pump.	a. Apply a warm compress to the breasts for 5 minutes before using the breast pump. b. Gradually increase the breast pump setting from L1 and find a comfortable suction level for use. c. If you still cannot pump milk easily, stop using the breast pump and consult a healthcare professional.
The suction power is very weak or there is no suction at all.	a. Reassemble the breast pump according to the steps in the "Assembly Method." Ensure that the main connector, horn cover, silicone cylinder, duckbill valve, and other components are properly and fully installed. b. Check if the main unit is fully pressed into the connector. c. Inspect whether the duckbill valve is fully closed and not damaged, and whether the horn cover fits snugly against the breast.
Even if the suction power is turned up, milk still can't be extracted.	a. Align the nipple with the opening inside the horn cover. Start with a low setting for multiple massage cycles and try to relax. b. Use a warm towel to apply heat and massage the breasts before using the massage mode to pump milk.
The machine doesn't work.	a. The device can be used when the power adapter is plugged in. Please charge it first before use, or use it while plugged in. If the device does not work properly when the adapter is plugged in, please contact the seller.

Breast Milk Storage/Heating

After extracting breast milk, securely close the storage cap or seal the storage bag, and it must be immediately placed in the refrigerator, either refrigerated or frozen.

	At room temperature	Refrigerator	Freezer
Freshly expressed breast milk	19°C to 26°C	< 4°C	-15°C
Storage time	4 hours to 6 hours	3 to 5 days	2 weeks

Note: This chart is applicable to full-term healthy infants and is for reference only. For more information, please consult the hospital or healthcare professionals.

Storage Method

- Breast milk will expand after thawing. Do not fill the bottles or storage bags completely. Leave a 1/4 space for expansion.
- Label the time of pumping on the bottles or storage bags.
- To avoid losing the nutrients in breast milk, thaw frozen breast milk in the refrigerator or in a stream of lukewarm water (water temperature should not exceed 37°C).
- Thawed breast milk can be warmed by placing it in water at 50°C.
- Do not thaw or heat frozen breast milk in a microwave or a pot of boiling water to avoid nutrient loss and scalding the baby.
- When feeding, gently shake the container to remix the separated milk.

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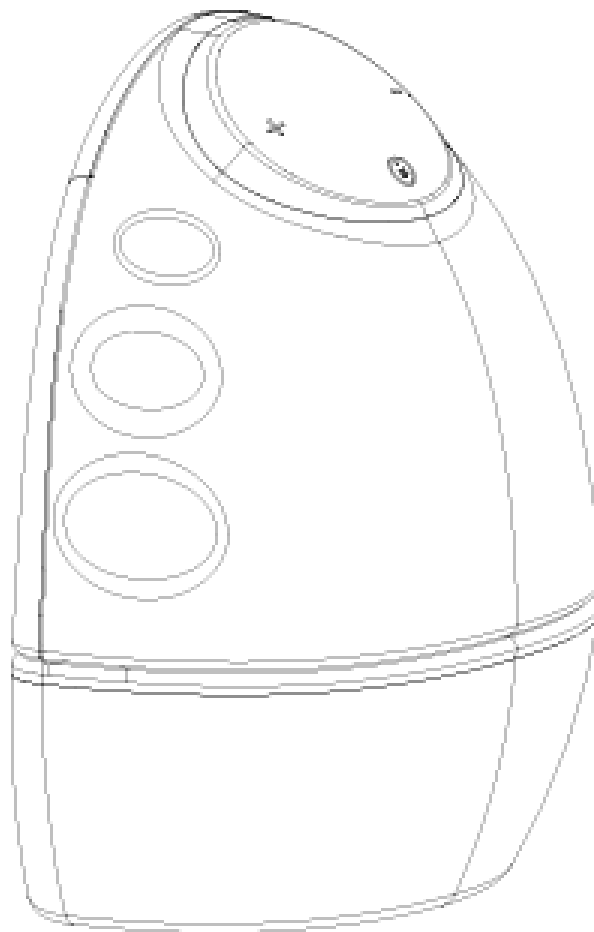
- Before the first use of the breast pump and before each subsequent use, you must disassemble all parts of the breast pump that come into contact with breast milk and then clean and sterilize them.
- Do not wash or sterilize the main body of the breast pump. You can simply wipe these parts clean with a soft cloth.
- You may also use a dishwasher (top rack only) or wash the necessary parts by hand in warm soapy water, then rinse thoroughly. It is recommended that you use a steam sterilizer or boil all parts that come into contact with breast milk for 5 minutes to sterilize them.

Caution

- When cleaning the duckbill valve and silicone cylinder, be very careful. If they are damaged, the breast pump will not work properly
- To clean the duckbill valve, gently rub it. Do not insert any objects, as this may cause damage.
- The main unit should not be washed with water. It can be wiped with a damp cloth that has been wrung out.
- Before use, ensure that there are no water stains inside the silicone cylinder to avoid damaging the main unit.

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