MAWe
by GADNIC

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BEBE0157

INDICE

ESPAÑOL.....	05
PORTUGUÊS.....	09
ENGLISH.....	13

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BEBE0157

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
LACTATION MASSAGER

BEBE0157

READ THE INSTRUCTIONS CAREFULLY BEFORE USE. KEEP THE MANUAL FOR FUTURE REFERENCE.


HOW TO USE

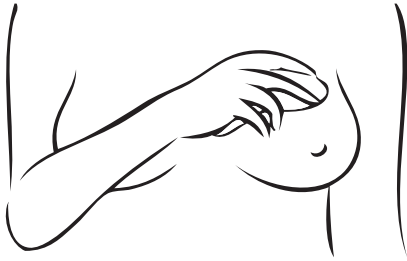Massage technique: Use the massage pad to gently rub the breast toward the nipple. This helps stimulate milk flow and relieve breast engorgement.

Positioning: Place the massage pad under the breast and massage upward toward the nipple.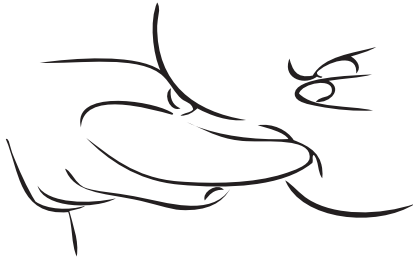

Gentle movement: Use the narrower end of the massager to make gentle circular movements on sensitive areas.

Relief for blocked ducts: Apply gentle pressure with the narrow end to areas with blocked ducts to help release blockages.


SAFETY PRECAUTIONS

1. Fully charge the device before using it for the first time.
2. Only use the included USB charging cable. Using other cables may damage the device or its circuitry.
3. Only clean it with a soft, damp cloth.
4. Store and charge at room temperature. Do not use or charge the massager if it has been exposed to external heat (e.g., left in a hot car or in direct sunlight).
5. Keep the device and manual in a dry, well-ventilated place.
6. Avoid exposing the massager to high temperatures or leaving it in hot environments.

PRODUCT FEATURES

MATERIAL	Liquid Silicone with Soft Touch
FUNCTIONS	Heat and Vibration
SIZE	51x25x110mm
WATERPROOF RATING	IPX7 (waterproof)
VIBRATION MODES	10 Frequency Options
HEATING SETTINGS	3 Heat Levels (45°C - 55°C)
CHARGING METHOD	USB DC Charging
USE TIME	Up to 1 Hour per Charge

BUTTON FUNCTIONS

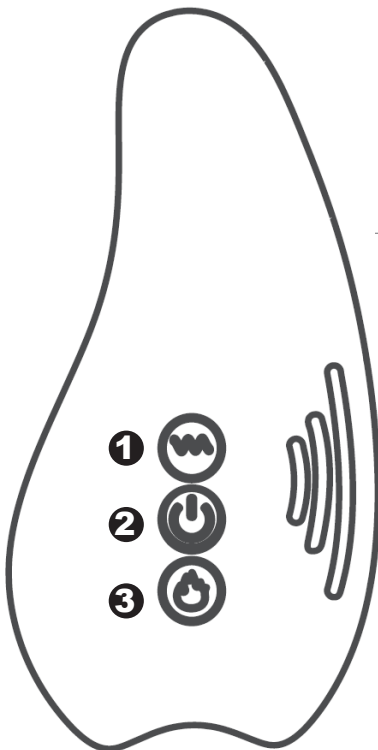- 1** Vibration Button
 - Press and hold for 2 seconds to activate vibration.
 - Short press to cycle through vibration modes.
 - Press and hold again to turn off.
- 2** Power Button (On/Off)
 - Press and hold for 2 seconds to power on/off.
 - Short press to toggle through available settings.
- 3** Heat Button
 - Short press once for low heat.
 - Press twice to activate medium heat.
 - Press three times to activate high heat.

Please read these instructions carefully before using your lactation massager.

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