GADNIC

MANUAL DE USUARIO IRRIGADOR NASAL

LIMNAS02

Índice

Español.....	2
Português.....	3
English.....	4

Limpiador nasal

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Nose Cleaner

How to use.

1. Make a cup of 500ml of warm water or saline water and fill it into the pot.
2. Tilt your head forward, open your mouth wide, breathe through your mouth, aim the nasal rinser spout at one nostril, release the air hole, the water will automatically flow into the nostril to clean the nasal cavity, and then flow out from the other nostril. If you need to stop, just seal the air hole with your index finger.
3. Please adjust the water outlet of the nasal rinser to the other nostril and repeat the above action.

Function:

1. Insisting on rinsing the nasal cavity every day can stimulate the sympathetic and parasympathetic nerves in the nasal cavity and improve the body's immunity.
2. Yoga people rinse the nasal cavity before sitting in meditation, which is conducive to opening the meridians, helping breathing and awakening the brow chakra.
3. Cleansing the dirt in the nasal cavity, cleaning the frontal sinuses, can also assist in the treatment of nasal diseases.

Precautions.

1. The use of the process must be open mouth, breathing through the mouth, not spring water to avoid choking
2. patients with nasal diseases, please use under the guidance of a doctor, if with medication rinse efficacy will be better.
3. Children should be used under adult supervision.

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