GADNIC

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Instructions for use:

How to use the mask:

1. Press the filter box cover with your hand and inhale gently. If you feel that the mask is slightly collapsed and close to your face, and there is no air leakage between the face and the mask, the air tightness is good.
2. If air leakage is detected, readjust the position of the mask and readjust the tightness of the headband to eliminate air leakage.
3. Repeat the above operations until it is airtight.
If the user cannot achieve a tight fit with the mask, do not enter the contaminated area.

How to use eye mask:

1. Clean your hands and make sure they are clean
2. Take out the eye mask and make sure it is clean and free of scratches
3. Place the eye mask above your eyes, and adjust the nose pads and headband to make the eye mask fit your face smoothly and closely.

Use:

This product is only used for respiratory protection against certain pollutants in the air. Improper use may cause illness or even death.

Instructions for use:

1. Failure to use it in accordance with the usage restrictions in this instruction manual, and/or failure to wear it continuously during contact with harmful substances, will reduce the protective function of this product and may lead to illness or even death;
2. Air pollutants that are harmful to the human body include tiny substances that cannot be seen by the naked eye;
3. If you smell or taste pollutants during use, or feel dizzy, irritated or otherwise uncomfortable, you should leave the polluted area immediately.
4. When not in use, respiratory protective equipment should be stored away from contaminated areas:
5. Dispose of used respiratory protective equipment in accordance with applicable regulations.

Usage restrictions:

1. This respiratory protective equipment cannot provide oxygen when used in filter mode. Do not use it in an anoxic environment with an oxygen content of at least 19.5%;
2. It is prohibited to modify, abuse or misuse this respiratory protective equipment.
3. It cannot be used when beards, facial hair and other factors will affect the good fit between the face and the mask.

Use time limit

1. If the respiratory equipment is damaged, leave the contaminated area immediately and repair or replace the respiratory protective equipment.
2. Replace the filter element according to the usage time limit of the filter element
3. Replace the poison filter disk according to the filter cartridge replacement schedule. If you smell or taste contaminants before this, or feel irritation, replace the poison filter cartridge immediately.
4. If the filter cotton is damaged, polluted or respiratory resistance increases, please replace the filter cotton.
Use 6 hours a day, recommended to replace after 20 days

Product specifications: MASKG002, gas mask and glasses set, black + yellow, 2 powder cakes Glasses material: glasses PVC, lenses PC

The filter element is activated carbon, composed of: cover body and replaceable filter element; used for anti-virus

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