GADNIC

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TAPONES DE OIDO
PARA DORMIR

PROTAU6X

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
Nominal diameter: / 10-13 mm, XS=10 S=11 M=12 L=13
Size M

SNR=28

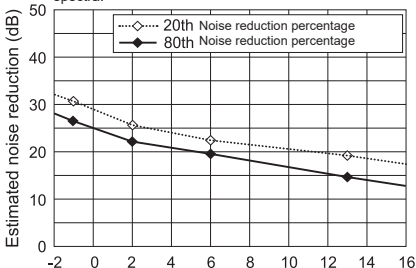Variability of noise reduction as a function of noise spectra.

Earplugs help you manage sound. Use them to protect against hearing damage from excessive decibels. The user must ensure that the earplugs are fitted, adjusted and maintained in accordance with this Instruction Manual. Any other use is not intended and therefore not permitted.

1. Insert the plug by grasping the plug between your thumb and forefinger. To facilitate insertion, pull gently on the top of your ear.
2. Gently push and twist until the tip of the earplug completely seals your ear canal.
3. Try multiple earplug tips to find the perfect fit.
4. Remove your earplugs using slow, twisting motions to gradually break the seal. Sudden or rapid removal of the hearing protector can cause damage to the eardrum.
5. Incorrect fitting, improper use or failure to follow the instructions for this device will decrease the effectiveness of noise reduction, increase the risk of hearing damage and affect the life of the reusable earplugs. The Hearing Protection Institute is not responsible for any damage resulting from improper use.
6. Check the working condition of the earplugs regularly and store them in the original, clean packaging. The silicone ring and tips are intended for repeated use and can be used approximately 100 times. Clean the plugs regularly with warm water and dry them after cleaning.

⚠ WARNING: Keep earplugs out of the reach of infants and young children, as they may become trapped in the windpipe and cause choking hazard. Use by children should be closely supervised by an adult.

⚠ CAUTION: Inspect the product before use and replace the earplugs if they are damaged or show signs of wear. In case of pain or discomfort while wearing the earplugs, discontinue use. If you continue to experience pain or discomfort, consult a physician. Do not force the earplugs into your ears.

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